ES: Manual para Pulsera de Prevención de Golpe de Calor
EN: Manual for Heat Stroke Prevention Wristband
FR: Manuel du Bracelet de Prévention du Coup de Chaleur
IT: Manuale per Braccialeto di Prevenzione del Colpo di Calore
PT: Manual da Pulseira de Prevenção de Golpe de Calor
GR: Εγχειρίδιο για Βραχιόλι Πρόληψης Θερμοπληξίας


www.biseif.com

CMZ-RF12


ES: La Pulsera de Prevención de Golpe de Calor es un dispositivo que detecta temperaturas corporales y ambientales anormales, y alerta mediante luz, sonido y vibración.
EN: The Heat Stroke Prevention Wristband is a device that detects abnormal body and environmental temperatures, and alerts by light, sound, and vibration.
FR: Le Bracelet de Prévention du Coup de Chaleur est un dispositif qui détecte les températures corporelles et environnementales anormales, et alerte par lumière, son et vibration.
IT: Il Braccialeto di Prevenzione del Colpo di Calore è un dispositivo che rileva temperature corporee e ambientali anomale e avvisa tramite luce, suono e vibrazione.
PT: A Pulseira de Prevenção de Golpe de Calor é um dispositivo que deteta temperaturas corporais e ambientais anormais e alerta através de luz, som e vibração.
GR: Το Βραχιόλι Πρόληψης Θερμοπληξίας είναι μια συσκευή που ανιχνεύει ανώμαλες θερμοκρασίες σώματος και περιβάλλοντος και ειδοποιεί με φως, ήχο και δόνηση.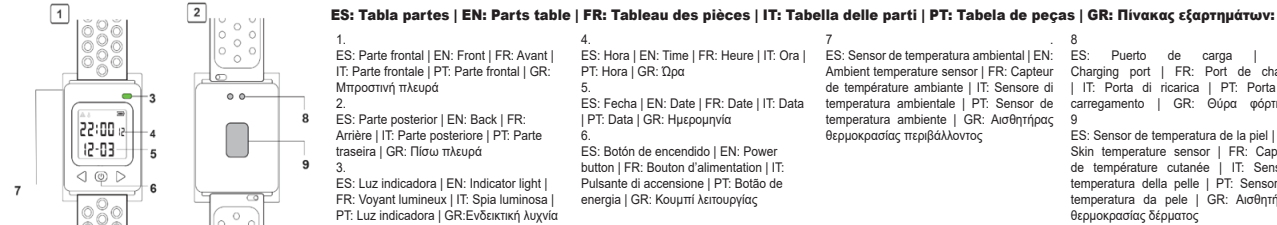

ES: Tabla partes | EN: Parts table | FR: Tableau des pièces | IT: Tabella delle parti | PT: Tabela de peças | GR: Πίνακας εξαρτημάτων:

- | | | | |
|--|--|--|--|
| 1. ES: Parte frontal EN: Front FR: Avant IT: Parte frontale PT: Parte frontal GR: Μπροστινή πλευρά | 4. ES: Hora EN: Time FR: Heure IT: Ora PT: Hora GR: Ώρα | 7. ES: Sensor de temperatura ambiental EN: Ambient temperature sensor FR: Capteur de température ambiante IT: Sensore di temperatura ambientale PT: Sensor de temperatura ambiente GR: Αισθητήρας θερμοκρασίας περιβάλλοντος | 8. ES: Puerto de carga EN: Charging port FR: Port de charge IT: Porta di ricarica PT: Porta de carregamento GR: Θύρα φόρτισης |
| 2. ES: Parte posterior EN: Back FR: Arrière IT: Parte posteriore PT: Parte traseira GR: Πίσω πλευρά | 5. ES: Fecha EN: Date FR: Date IT: Data PT: Data GR: Ημερομηνία | | 9. ES: Sensor de temperatura de la piel EN: Skin temperature sensor FR: Capteur de température cutanée IT: Sensore di temperatura della pelle PT: Sensor de temperatura da pele GR: Αισθητήρας θερμοκρασίας δέρματος |
| 3. ES: Luz indicadora EN: Indicator light FR: Voyant lumineux IT: Spia luminosa PT: Luz indicadora GR: Ενδεικτική Λυχνία | 6. ES: Botón de encendido EN: Power button FR: Bouton d'alimentation IT: Pulsante di accensione PT: Botão de energia GR: Κουμπιέ λειτουργίας | | |

ES: Operaciones básicas | EN: Basic Operations | FR: Opérations de base | IT: Operazioni di base | PT: Operações básicas | GR: Βασικές λειτουργίες

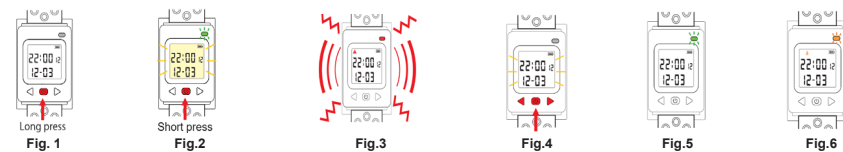Biseif Safety Solutions, S.L.
B56046964
c.Imprenta Alborada, 110
14014 Córdoba (Spain)
Made in China
91440300326682865Q

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EN | CE and WEEE Declaration of Conformity (Waste Electrical and Electronic Equipment)
Biseif Safety Solutions, S.L. hereby declares that this product complies with the essential requirements and other relevant provisions of Directive 2014/30/UE and all other applicable UE Directives. This WEEE symbol indicates that this product must not be disposed of as household waste but should be delivered to an appropriate collection point for recycling. Proper disposal and recycling help protect natural resources, human health, and the environment. For more information on the disposal and recycling of this product, contact your local municipality, waste management service, or the shop where you purchased it.

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EN | Instructions for Use

How to wear the wristband: Before wearing the wristband, ensure that the skin is clean and dry | Avoid applying creams, oils or lotions on the contact area as they may affect measurement | If you exercised or were exposed to sunlight, wait a few minutes until your body temperature stabilizes before putting on the device | Place the wristband on the non-dominant wrist (usually the left if right-handed), one or two fingers above the

wrist bone | Ensure the inner sensor is in direct contact with the skin | Avoid placing it over clothing, bandages, or accessories | Adjust the strap so the wristband fits firmly but comfortably | How to turn on/off (see fig.1): Turn on: Press and hold the power button for 3 seconds | Turn off: Press and hold the power button for 3 seconds until the red light turns off, then release | Normal working mode (see fig.2): Green light flashes once every 90 seconds | Screen: battery level, time, date, and temperature warnings icons | Middle button: short press to activate the backlight (lasts 10 seconds)

| With backlight on, a short press activates quick detection (changed from 90 seconds to 15 seconds) | Left/right button: time and date adjustment | How to interpret readings or alarms | Body temperature alarm (see fig.3): If the body temperature exceeds 37.8°C, the device activates alarm mode for 1 minute: red light flashes 3 times per second, buzzer beeps 3 times per second, motor vibrates 1 second then pauses 1 second | How to stop the alarm: When body temperature drops below 37.8°C | Removing the wristband | Pressing the power button to turn off the device | Ambient

POWERED BY KALFIT

temperature alarm (**see fig.6**): If ambient temperature reaches 42°C or higher, alarm activates for 1 minute: orange light flashes 3 times per second | How to stop the alarm: When ambient temperature drops below 42°C | Pressing the power button to turn off the device | Low battery alarm: If voltage is below 3.3V, red light flashes to indicate low battery | How to stop the alarm: Charge the battery with the device on until voltage exceeds 3V | Charge with device off | Pressing the power button to turn off the device | How to charge (**see fig.5**): Use the included USB cable | Green light remains on while charging | Green light turns off when fully charged | Date and time setting (**see fig.4**): Quickly press the middle button three times | Set month, year, hour, and minute (24-hour format) | Use left/right buttons to adjust and middle button to confirm | Long-press middle button for 2 seconds to save and exit | Interface closes after 15 seconds of inactivity without saving changes.

EN | Contents | Heat stroke prevention wristband, charging cable, instruction manual. **EN | Maintenance and Cleaning** | Keep the bracelet clean and dry after each use. | Avoid exposing the device to extreme temperatures (below -20°C or above 80°C). | Do not leave the bracelet under direct sunlight for long periods (for example, on a car dashboard). | Avoid prolonged contact with sweat, lotions, oils, or cosmetics, which could damage the materials. | Check regularly that the sensor and charging connectors are free of dirt or corrosion. | If the device will not be used for a long time,

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charge it to about 50% and store it turned off (to preserve battery life).

EN | Safety Instructions | Due to the material characteristics, moisture, sweat, friction, and UV rays may cause discoloration. | Do not drop or hit the device sharply to avoid malfunctions or damage. | Dirt on the sensor or improper use may affect measurement accuracy. Use and maintain the device properly. | This product does not prevent heat stroke: it is a preventive device. Rest adequately and consult a doctor if you feel unwell.

EN | Safety Warnings | This device is designed for personal use as a wellness or environmental monitoring tool. | It is not a medical device and does not replace professional advice, diagnosis, or treatment. | It should not be used to monitor medical conditions or for clinical decisions. | If the device overheats abnormally, stop using it and contact technical support. | Do not immerse the device in water or use it during aquatic activities unless a specific IP protection rating is indicated. | Avoid contact with liquids, steam, or very humid environments. | Always dry the device well before charging or using it. | Use only the charging cable and adapter recommended by the manufacturer. | Do not charge the device near heat sources or flammable materials. | Do not handle, pierce, or expose the battery to fire or heat. | Do not charge the device if the casing or connector is wet or damaged. | Do not attempt to open, modify, or repair the device yourself. | If unauthorized handling will void the warranty, if

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irritation, redness, or discomfort occurs, stop using it until symptoms disappear. | Do not use the device on wounds, burns, or inflamed areas. | This product contains electronic components that may be affected by strong electromagnetic fields (e.g., medical equipment, microwave ovens, or powerful transmitters). | Keep the device away from pacemakers or other implanted medical devices (at least 15 cm distance). | Not suitable for children under 3 years old. | Keep the device, accessories, and charging cable out of reach of children. | Users with sensory or cognitive disabilities should use it under supervision. | Measurements may be affected by environmental conditions (wind, humidity, direct sunlight, sweat). | The accuracy of readings may vary depending on fit or sensor position. | Heat stroke alerts are indicative and do not guarantee prevention of thermal injuries. | In case of dizziness, confusion, or fainting, seek immediate medical attention. | The manufacturer is not responsible for damage or injury caused by improper use or failure to follow safety instructions. | This product is not certified for explosive environments (ATEX). | This product has no communication certification and is not subject to GDPR regulations. **EN | Warranty and After-Sales Service** | For any problem with warranty or after-sales service, please contact your distributor.

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